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User manual - Disinfectants and soap dispenser

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GB

Congratulations!

You bought an approved dispenser which is easy to handle and assures you working processes as used.

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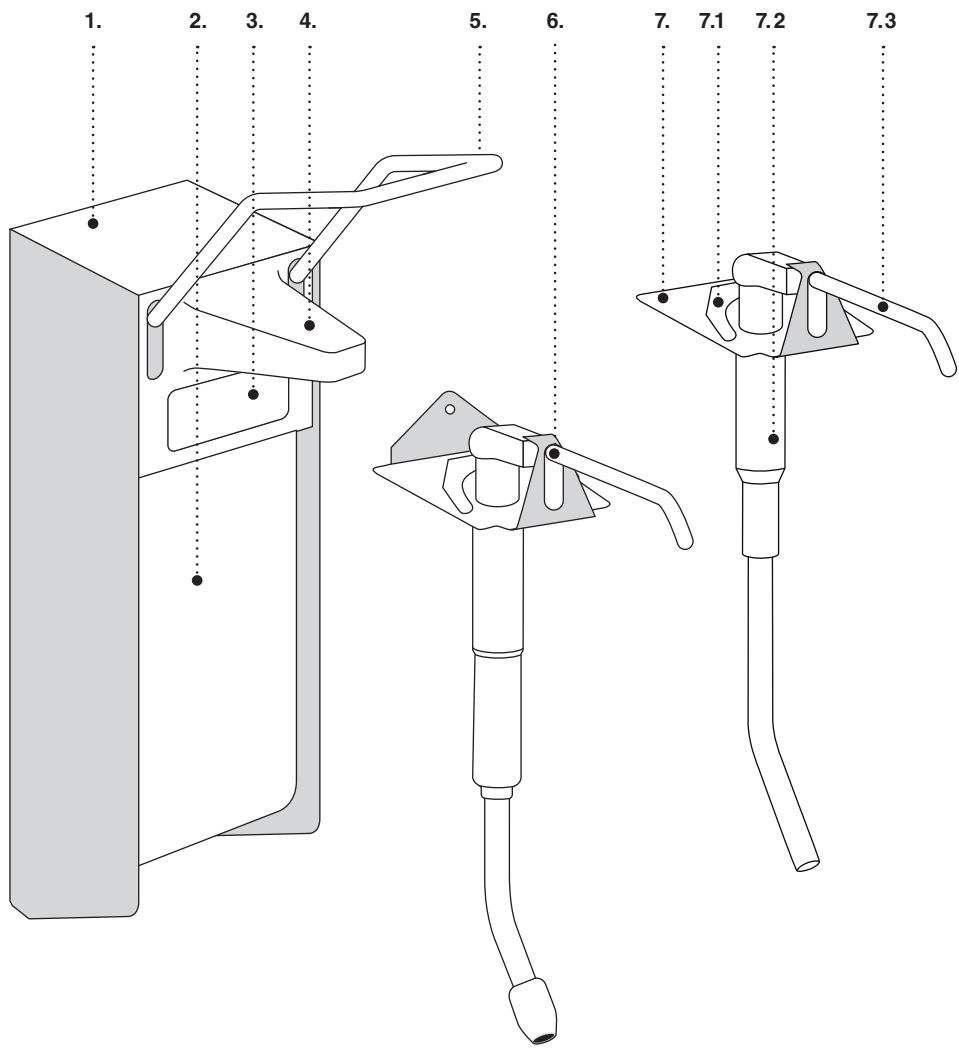
GB

1. Aluminium housing
2. Container
3. Marking clip
4. Outlet aperture
5. Stainless steel lever
6. Universal pump stainless steel
7. Universal pump plastic
- 7.1 Dosing fork
- 7.2 Suction tube
- 7.3 Spout

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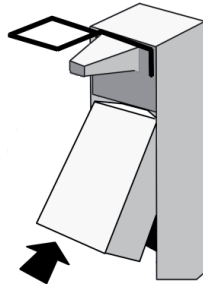
GB

1. Mounting of Dispenser

Before mounting, do not forget to define the right position of the dispenser. Fix it above a wash basin.

- Above the dispenser:
keep a space of minimum 40 mm
- Below the dispenser:
min. 150 mm for 500 ml dispenser
min. 200 mm for 1.000 ml dispenser
- Lift up bottle lightly and pull it downward out of the body.
- Press locking on the wall plate and pull wall plate downward out of the body.
- Insert dowels in predrilled holes and fix the wall holder with screws.
- Push dispenser body top down over wall holder until you hear the locking snap in.
- Push full bottle bottom-up over the dispenser pump and insert in the body.
- When first used press down the lever several time until liquid leaks from the pump head. Catch this drop with a convenient pot.
- The dispenser is ready for use now

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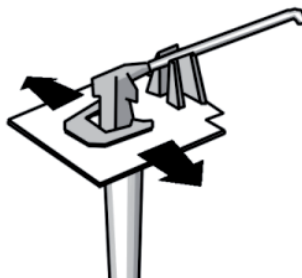
2. Important notice

Dosing of liquid:

Delivery is with a max. volume of 1,8ml per pass. By changing the dosing fork you can reduce the volume.

By pushing right: approx. 1,2ml per pass.

By pushing left: approx. 0,8ml per pass.

Autoclaving pump or complete dispenser:

Proceed as described in step 1 and 2. Autoclave the dispenser using hot steam (121°C at a pressure of 1 bar).

Attention: The pure plastic pump is not autoclavable.

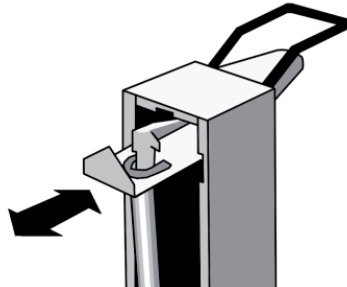
Maintenance:

The dispenser is basically maintenance free.

On the recommendations of hygiene and prevention of infections the dispensers and the pumps should be cleaned and disinfected regularly:

- For cleaning the pump first remove bottle, then press locking at the wall plate and pull dispenser upwards.
- Hold lever in upper position and pull out the pump backwards. For inserting again go reverse.
- Use gentle cleaning agents only.
- If the pump is blocked (e.g. no use for a longer time), remove bottle and rinse thoroughly with hot water until the work is perfect again.

Note: Although the stainless steel pump is made of high quality materials, there may be certain soaps to rust-colored surfaces. In this case, we recommend the use of plastic pumps.

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